

# EyeBrid™

## Silicone Multifocal

### Silicone Hydrogel hybrid contact lens

## Indications

- Presbyopia
- High ametropias, astigmatisms
- Wearers of RGPs (comfort/centration/stability problems)

## Technical information

Material	100 dk + UV blocker Silicone hydrogel
$\varnothing_T$	14.90 mm (15.50 mm possible)
$\varnothing_{RGP}$	10.00 mm (8.50 mm possible)
Base curve BC	From 5.50 to 10.00 mm in 0.05 mm steps Back toricity from 0.50 to 1.80 mm in 0.05 mm steps
Soft skirt J	J 0.0 (standard skirt) From -1.0 to +1.0 in 0.5 steps
Power P	From -40.00 to +40.00 $\delta$ in 0.25 $\delta$ steps Cyl. from -0.50 to -6.00 $\delta$ in 0.25 $\delta$ steps All axis
Additions	From +0.75 to +3.00 $\delta$ in 0.25 $\delta$ steps


## 2 designs

### N lens (Near)

Central near vision

### D lens (Distance)

Central distance vision

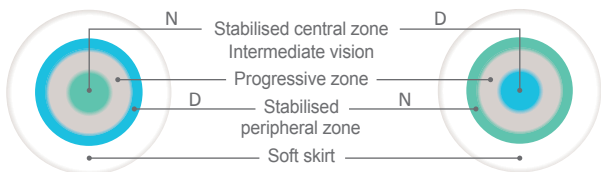

## Wear and care

Wear	Bi-annual replacement, daily wear
Care	Oté Clean (cleaning) + Multipurpose or peroxide
Handling	Instill 2 drops of hydration solution at lens insertion Remove with a contact lens remover or like a RGP

